



Cardiac Science Powerheart G3 & G5



1. Open the lid.
2. Wait for the AED to indicate status. Observe the change of the status indicator to **RED**.
3. After approximately five seconds, verify that the status indicator returns to **GREEN**.
4. Check expiration dates on the electrodes/pads.
5. Listen for the voice prompts, and check that the display shows the proper corresponding text prompts.
6. Close the lid. Observe the change of the status indicator to **RED**.
7. After approximately 5 seconds, verify that the status indicator turns to **GREEN**.
8. If any of the following items need attention or replacement, indicate so in the “Notes” field of your online form. These notes will be submitted automatically to EHS for attention.
 - AED accessory kit
 - Electrodes/pads
 - Contact card
 - Incorrect location
9. Complete online form found [here](#). NOTE: The form includes extra steps for other AED models. Be sure to check the boxes indicated for Cardiac Science.

Environmental Health & Safety
4472 TAMU
College Station, TX 77843-4472
Tel. 979.845.2132
Fax. 979.845.1348

<http://ehsd.tamu.edu>

If you have any questions, please contact AED@tamu.edu.